

# Gelfilm®

absorbable gelatin  
film, USP

PHARMACIA

## DESCRIPTION

GELFILM Sterile Film and GELFILM Sterile Ophthalmic Film are absorbable gelatin film approximately 0.075 mm in thickness, designed for use as an absorbable gelatin implant in neurosurgery and thoracic and ocular surgery.

In the dry state absorbable gelatin film has the appearance and texture of cellophane of equivalent thickness; when moistened, it assumes a rubbery consistency and can be cut to desired size and shape and fitted to rounded or irregular surfaces.

Gelfilm  
absorbable gelatin  
film, USP



## ACTIONS

Rate of absorption of GELFILM Sterile Film and GELFILM Sterile Ophthalmic Film after implantation ranges from one to six months depending on size of the implant and site of implantation. Pleural and muscle implants have been reported to be completely absorbed in eight to 14 days, whereas dural and ocular implants usually require at least two to five months for absorption. Absence of undue tissue reaction incident to implantation and absorption of absorbable gelatin film, with consequent decreased likelihood of developing adhesions, has been found to be of particular value in dural and ocular implants.

## INDICATIONS AND USAGE

**Neurosurgery:** Nonconductive to undue inflammatory reaction and absorbable at a rate sufficiently slow to permit dural regeneration and healing of the arachnoid layer, GELFILM Sterile Film favorably meets requisites for a dural substitute. Use in patients undergoing craniotomies has been reported to prevent development of meningocerebral adhesions and thereby reduce risk of postoperative sequelae.

**Thoracic Surgery:** In repair of pleural defects in connection with thoracotomies, thoracoplasties, and extrapleural procedures, implantation of GELFILM Sterile Film has been observed to be followed by minimal tissue reaction and closure of the defect by ingrowth of regenerating pleural and fibrous tissue across the gradually resorbed GELFILM implant.

**Ocular Surgery:** Various ocular surgical procedures in which GELFILM Sterile Ophthalmic Film has been used include glaucoma filtration operations (ie, iridencleisis and trephination), extraocular muscle surgery, and diathermy or scleral "buckling" operations for retinal detachment. Experimental studies in rabbits and clinical trials in patients have shown a remarkable lack of cellular reaction to GELFILM implanted subconjunctivally or used as a seton into the anterior chamber. Objective evidence that GELFILM implants aid in preventing formation of adhesions between contiguous ocular structures has been reported as follows: in iridencleisis in which GELFILM was employed as a seton, the resultant filtrating areas were large and there was no postoperative rise in intraocular tension; in extraocular muscle surgery and operations for retinal detachment, insertion of GELFILM implants between contiguous tissue layers was found to enhance the ease of secondary operations.

**Directions For Use:** To prepare for use, immerse absorbable gelatin film in sterile saline solution and allow it to soak until it becomes quite pliable; it may then be cut to desired size and shape without difficulty and applied as follows:

*For covering dural defects,* GELFILM Sterile Film is placed over the surface of the brain, the edges of the implant tucked beneath the dura and the wound then closed in the usual manner. If desired, the GELFILM can be sutured loosely to the dura. Care must be exercised, however, because moist film tears easily. *For covering pleural defects,* GELFILM Sterile Film is placed over the defect and anchored in place by small interrupted sutures.

*For use as a seton in iridencleisis,* a small piece of GELFILM Sterile Ophthalmic Film (approx. 4 x 10 mm) is placed over the prolapsed iris pillar

# Gelfilm

brand of absorbable gelatin film, USP

parallel to the limbus; Tenon's capsule and the conjunctiva are then closed with continuous absorbable sutures spaced to insure tight closure. *In diathermy or scleral "buckling" operations,* GELFILM Sterile Ophthalmic Film may be placed over the sclera, the muscle and the conjunctiva then sutured over the underlying GELFILM. *In extraocular muscle surgery,* GELFILM Sterile Ophthalmic Film may be placed over and beneath the muscle before Tenon's capsule and the conjunctiva are closed in layers.

## CONTRAINDICATIONS

None known.

## PRECAUTIONS

Because the rate of absorption of GELFILM Sterile Film and GELFILM Sterile Ophthalmic Film is likely to be increased in presence of purulent exudation, it is recommended that absorbable

(continued below)

gelatin film not be implanted in grossly contaminated or infected surgical wounds.

## HOW SUPPLIED

GELFILM Sterile Film and GELFILM Sterile Ophthalmic Film are supplied in the following packages:

GELFILM Sterile Film, for use in neurosurgery and thoracic surgery, sterile envelopes, one per carton, 09-0283-01.

GELFILM Sterile Ophthalmic Film, for use in ocular surgery, sterile envelopes, six per carton, 09-0297-01.

## STORAGE AND HANDLING

GELFILM Sterile Film and GELFILM Sterile Ophthalmic Film should be stored at 25°C (77°F); excursions permitted to 15-30°C (59-86°F) [see USP Controlled Room Temperature]. Once the envelopes have been opened, contents are subject to contamination. To insure sterility, it is recommended that absorbable gelatin film be used immediately after withdrawal from the envelope.

**Caution:** Federal law restricts this device to sale by or on the order of a physician.



EU Authorised Representative  
Pharmacia  
Puurs Belgium

Pharmacia & Upjohn Company  
A subsidiary of Pharmacia Corporation  
Kalamazoo, MI 49001, USA

Revised March 1999

811 344 410  
691320

PHARMACIA

Black

COMPOSITION ORDER # 28640	PRODUCT GELFILM	COPY CODE # 811 344 410		
CCS # 029703-000	NDC #	EDP # 691320		
ITEM Insert	SIZE 2.5 x 12"	FOLDED SIZE 2.5 x 1"	DRAWING # PD1779	REVISION # 1
ADDITIONAL INFORMATION		DATE 8-22-05	TYPESET BY KL	

PHARMACIA

Black

COMPOSITION ORDER # 28640	PRODUCT GELFILM	COPY CODE # 811 344 410		
CCS # 029703-000	NDC #	EDP # 691320		
ITEM Insert	SIZE 2.5 x 12"	FOLDED SIZE 2.5 x 1"	DRAWING # PD1779	REVISION # 1
ADDITIONAL INFORMATION		DATE 8-22-05	TYPESET BY KL	